

ISOLATOR® PLUS

N-95 PARTICULATE RESPIRATOR HIGH RISK FACE MASK



A NIOSH approved N95 Particulate Respirator that meets guidelines for TB exposure control. Indicated for use when treating patients with airborne diseases such as TB and Influenza.

GPRN95
28/Box



Donning Instructions:



1. Grasp nosepiece at top of mask with one hand. Slide finger of other hand behind chinpiece.



2. Open mask by pulling down and out on chinpiece until mask is fully open. It is IMPORTANT that mask is fully open.



3. Place chin in pocket created. Stretch bands over back of head, positioning bottom band behind neck below ears and top band near crown of head.



4. Conform nosepiece to nose and face contours making sure Support Arch is fully extended away from nose and mouth.



5. Holding mask at nosepiece, place one finger inside the mask on the malleable aluminum chinpiece, pull down gently so that mask is snug against face.



6. Pinch any excess material together under chin and fold over or twist.



7. Face Fit Check:

To check fit, place both hands along the edges of the respirator and exhale. If air escapes around your nose, adjust the nosepiece. If air leaks at the respirator edges, unfold or untwist the chinpiece, then repeat steps 5 and 6.



8. N95 Respirator correctly fit to face.



! WARNING:

1. This product does not eliminate the risk of contracting any disease or infection. Change immediately if the respirator comes in contact with blood or fluids. OSHA has not set a permissible exposure level for airborne biohazards.
2. Follow all instructions and warnings on the use of this respirator. Failure to do so will reduce respirator effectiveness, wearer protection and may result in sickness or death. Wear during all times of exposure. For additional information refer to your supervisor and/or see dispenser box.
3. Before use, the wearer must be trained by the employer in proper respirator use in accordance with applicable safety and health standards for the contaminant and exposure level in the assigned work area.
4. See fitting instructions at left.
5. The preceding warnings have been approved by the U.S. Department of Health & Human Resources and NIOSH.



Crosstex International
10 Ranick Road
Hauppauge, NY 11788
631-582-6777



This respirator is approved only in the following configuration:

TC-	Protection ¹	Respirator #	Cautions and Limitations ²
84A-0457	N-95	GPRN95	A B C J L M N O

¹Protection:

N-95 - Particulate Filter (95% filter efficiency level) effective against particulate aerosols free of oil; time use restrictions may apply.

²Cautions and Limitations:

- A. Not for use in atmospheres containing less than 19.5% oxygen.
- B. Not for use in atmospheres immediately dangerous to life or health.
- C. Do not exceed maximum use concentrations established by applicable regulatory standards & guidelines as may be in effect from time to time.
- J. Failure to properly use and maintain this product could result in injury or death.
- L. Follow the manufacturer's instructions for changing cartridges and/or filters.
- M. All approved respirators shall be selected, fitted, used, and maintained in accordance with MSHA, OSHA, and other applicable regulations or guidelines as may be in effect from time to time.
- N. Never substitute, modify, add, or omit parts. Use only exact replacement parts in the configuration as specified by the manufacturer.
- O. Refer to user instructions and/or maintenance manuals for information on use and maintenance of these respirators.

LIMITED WARRANTY: IMPORTANT NOTICE TO PURCHASER

This limited warranty is made in lieu of the warranties of merchantability, fitness for particular purposes and all other warranties, expressed or implied. There are no other warranties which extend beyond the description on the face hereof. The physical standards and specifications of Crosstex International will be met by products sold.

Exclusive Remedies: Damages for the breach of this limited warranty are limited to the replacement of such quantity of Crosstex International products proved to be defectively manufactured. Except as provided above, Crosstex International shall not be liable or responsible for any loss, damage, or liability, direct, indirect, incidental, special, or consequential, arising out of sale, use, or misuse, or the inability to use products by the user.

Crosstex International can make no warranties or representations, either expressed or implied, that these products will fully protect the user from exposure to blood or body fluids or risk of contracting infectious diseases. OSHA requires the employer to evaluate the anticipated exposure and select the appropriate protective masks to prevent contamination of skin, eyes and respiratory passages.



MADE IN
USA



CROSSTEX
A CANTEL MEDICAL COMPANY

PROTECTS®