



Cements



CEMENT DURATION CHART

REQUEST INFO

	MIXING	WORKING	SETTING
HY-BOND CEMENT			
POLYCARBOXYLATE	within 60 sec	3 min start of mixing	6 min end of mixing
POLYCARBOXYLATE TEMPORARY	within 60 sec until homogenous	-	5 min end of mixing
ZINC OXIDE EUGENOL TEMPORARY	within 60 sec	2 min end of mixing until crown is seated	5 min end of mixing
ZINC PHOSPHATE	within 60 sec	3 min/30 sec start of mixing	6 min/30 sec end of mixing
RESI GLASS	10 sec	3 min/30 sec start of mixing	7 min end of mixing
GLASS IONOMER CEMENT			
BASE	not to exceed 30 sec	2 min/30 sec start of mixing	4 min end of mixing
CORE SHADE	not to exceed 30 sec	2 min/50 sec start of mixing	4 min end of mixing
CX-PLUS	30 sec	3 min/30 sec start of mixing	3 min/45 sec end of mixing
CX-PLUS CAPSULES	10 sec	3 min/10 sec start of mixing	4 min/10 sec end of mixing