


Now available with reflective backing!
Try our new Reflective DryTips®


 Video and instructions for use available at www.microbrush.com/drytips

Ultimate Saliva Control

Far outlasts cotton rolls and other absorbent pads, and doesn't lose moisture when handled.

The DryTips® advantage over traditional cotton rolls:

- Complete coverage of parotid orifice and entire buccal mucosa with polyethylene film for cutting protection
- Facilitates work far back in the oral cavity
- Flexible; accommodates to cheek movements
- Comfortably stays in place, yet is easily removed with water spray
- Reflective coating for increased visibility and to aid in retraction

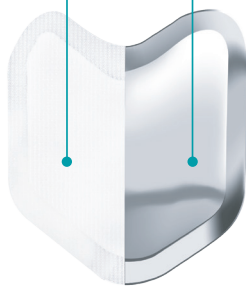

NEW! Reflective coating now available.

Ideal for:

- Impressions
- Occlusal Adjusting
- Orthodontics
- Paint-on techniques
- Composites
- Cementation
- Fissure sealing
- Fluoride applications

DryTips® Ordering Information:

ITEM NUMBER	SIZE	QUANTITY	COLOR
291542	Large	50	White
291543	Small	50	White
291726	Large	50	Silver
291727	Small	50	Silver

Order today through your preferred dealer.

For questions, call Microbrush® International at (866) 866-8698.

Your Choice of 3 Great Offers!

BUY 2

Two 4-packs of the Microbrush® Plus Series or Tube Series Applicators

GET 1

One Box of New! Reflective DryTips for no charge



BUY 1

One Refill Pack of Coated or Uncoated ContactPro® SmartBand Matrix Bands

GET 1

One Refill Pack of the same ContactPro SmartBand Matrix Bands for no charge*

*See side 2 for product details



MUST WRITE Promo Code: Q118 on Invoice

BUY 3

Three Boxes of DryTips® or Reflective DryTips®

GET 1

One Box of Reflective DryTips for no charge



ContactPro SmartBands can be used with any matrix ring!

Full Name		Date of Purchase	
Street Address			
City	State	Zip	
E-Mail	Telephone	Fax	

*To redeem this offer, send a copy of your qualifying invoice to: freegoods@microbrush.com or fax it to 888-771-7573. Purchases must be shown on one invoice. Offer limited to one per office and must be received no later than one month after the promotion expires. Offer good 1/1/2018-3/31/2018.

CONTACT PRO[®]

MATRIX SYSTEM



See the ContactPro[®] matrix system in action at microbrush.com/contactpro

Simple Technique for Outstanding Results

The reliable, consistent and easy to use system that provides solutions for Class II Restorations.

1 Place the Matrix Band



Select a band that matches the height of the tooth and place it with the concave edge toward the occlusal margin using the SmartTab at the top of the band.

2 Place the Wedge



Place a finger on the band to prevent dislodgment and firmly insert wedge to seal the gingival margin.

3 Place the Ring



Open the ring with your ContactPro[®] Ring Forceps and place it directly over the wedge, seating it as far gingivally as possible.

4 Voila!



Simple technique for outstanding results.

CONTACT PRO[®] SmartBand

Ultra-thin stainless steel for contouring to the tooth

Naturally curved for easy use

SmartTab feature for easy pick-up and placement

Defined marginal ridge

Can be used with any matrix ring!

Non-stick coated bands available for easy placement and removal

ContactPro[®] SmartBand Matrix Bands Refill Pack Ordering Information:

ITEM NUMBER	SIZE	QUANTITY	COLOR
291704	3.5mm	100ct	White
291591	4.5mm	100ct	Green
291593	5.5mm	100ct	Blue
291705	6.5mm	25ct	Purple
291706	7.5mm	25ct	Gray
291707	3.5mm	100ct	Silver
291590	4.5mm	100ct	Silver
291592	5.5mm	100ct	Silver
291708	6.5mm	25ct	Silver
291709	7.5mm	25ct	Silver

Available in Uncoated or Coated in Multiple Sizes!



Order today through your preferred dealer. For questions, call Microbrush[®] International at (866) 866-8698.